



Mounting instructions:

1. The seat is usually mounted at a height of 500 mm. This is also recommended for disabled people.
2. Drill holes for screw type suitable for metal, wood or expanderscrew (7mm hole), in concrete or brick.
3. In light concrete the seat is mounted using through-bolts and nuts.
4. Mount using 50x5 nickel-plated round-headed countersunk screws in timber walls or laths.
5. Position the seat and mark the holes. Screw in the screws with prepared holes, fold down the seat and check the movement. If necessary, adjust the seat. The two outer screws are mounted (where a washer is sometimes necessary).
6. If necessary a strengthening plate may have to be used.

